

HEEL PAIN THERAPY

**OVERNIGHT
RELIEF**

THE TWO MOST COMMON CAUSES OF HEEL PAIN ARE: PLANTAR FASCIITIS AND ACHILLES TENDONITIS.

This drug-free treatment of pulsed electromagnetic therapy is used by physicians, therapists and athletic trainers worldwide to reduce pain and swelling and accelerate healing. Now through embedded microchip technology this office based therapy is available in a convenient, safe and easy to use, low-cost patch.


HOW ACTIPATCH™ WORKS: ActiPatch™ induces an undetectable electrical current into damaged cells at a frequency which recharges the cell, stops the release of pain and inflammatory mediators, increases blood flow and re-establishes normal cell interaction. As a result, your body can begin to heal faster and you experience less swelling, pain and bruising.

FOR BEST RESULTS Apply the ActiPatch™ immediately after surgery or injury. Wear continuously, a minimum of 6 hours, until pain and swelling are gone. ActiPatch is safe to use 24 hours per day and will deliver an estimated 360 hours of treatment. Remove ActiPatch before bathing or showering. Store ActiPatch in a cool dry location.

COMFORTABLE NIGHT TIME THERAPY!

- ✓ Easy to wear especially at night to relieve pain and inflammation.
- ✓ Each device lasts 15 days
- ✓ Drug-free, safe and affordable.
- ✓ Localized treatment right where you need it... No side effects.
- ✓ Patients and doctors alike love ActiPatch!

PLANTAR FASCIITIS is inflammation of the tissue band connecting the heel bone to base of the toes. The pain is centered under the heel and usually flares up with the first steps in the morning after resting overnight.

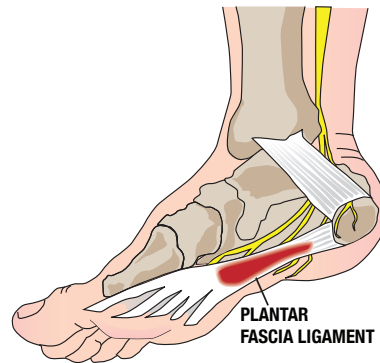

ACHILLES TENDONITIS is an inflammation of the tendon that links your leg muscles to the back of your heel bone. This inflammation causes pain in the back of the heel and may cause the skin to thicken, get red and swell.



\$39.95
USD Retail

U.S. & Foreign Patents Pending

CE 0470 **EC REP** **LATEX** All Components Latex Free

i Read instructions before using

1-877-757-2284

Manufactured by: BioElectronics Corporation
4539 Metropolitan Court :: Frederick, MD 21704
Stock Symbol: BIEL:PK :: www.BioElectronicsCorp.com